

INSTRUCTION FOR USE

Premature Baby Incubator Disinfectant



PRODUCT NAME : Premature Baby Incubator Disinfectant

TRADEMARK : GBL® SURFACTO™ DIAPP

PRODUCT REF NO : 6001

| PACKING TYPE | ORDER NO |
|---------------------------------|--------------|
| HDPE; 1 Liter Bottle with spray | 436001051000 |
| HDPE; 5 Liter Canister | 436001015000 |

INTENDED USE

GBL® SURFACTO™ DIAPP is effective for cleaning and disinfecting the interior and exterior surfaces of infant incubators and bassinets, exterior surfaces of anesthesia machines and respiratory therapy equipment surfaces, operating room tables and lights which are made of plastic, stainless steel, plexiglass, and glass.

PRODUCT SPECIFICATION

| | |
|------------------|-----------|
| Form | Liquid |
| Water Solubility | Soluble |
| Color | Colorless |
| Odor | Odorless |
| pH @ 25°C (ca) | 6.5±0.2 |

COMPOSITION

isopropyl alcohol (7 %), ethanol 96 %, Benzethonium chloride (2'-2-(4-Diisobutylphenoxy)ethoxy)ethyl) dimethylbenzylammonium chloride (0,3%), corrosion inhibitor, antioxidant, distilled water. Free of benzalkonium chloride and other quaternary ammonium compounds, aldehyde and phenol, buffers, and other alcohols.

MICROBIOLOGICAL PROPERTIES

| Effect | European Standard | Microorganisms (with ATCC) | Concentration of use | Contact time |
|----------------|-------------------|--|----------------------|--------------|
| Bactericidal | EN 13727 | Staphylococcus aureus 6538 Pseudomonas aeruginosa 15442 Enterococcus hirae 10541 | Ready-to-use | 3 min. |
| Fungicidal | EN 13624 | Candida albicans 10231 Aspergillus brasiliensis ATCC 16404 | Ready-to-use | 3 min. |
| Sporicidal | AOAC | Bacillus spizizeni ATCC 6633 | Ready-to-use | 60 min. |
| Tuberculocidal | EN 14348 | Mycobacterium terrae ATCC 15755 | Ready-to-use | 60 min. |

HOW TO USE

Spray on the surface. Wait for 3 minutes. Wipe and dry with a sterile wipe.

WARNINGS

Flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces. No smoking. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin.

STABILITY AND STORAGE












Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

| Storage Temperature | Shelf Life | Usage Time After Opening the Cap |
|---------------------|------------|----------------------------------|
| 5/25 °C | 2 years | 3 months |

PACKAGING

| Product | Packaging Type | Packaging Units |
|----------------------|---------------------------------|-----------------|
| GBL® SURFACTO™ DIAPP | HDPE; 1 Liter Bottle with spray | 1L x 20 |
| GBL® SURFACTO™ DIAPP | HDPE; 5 Liter Canister | 5L x 2 |

LABEL SYMBOL KEYS

| | | | |
|---|--|---|--|
|  Catalog Number |  Production Date |  Storage Temperature |  Manufacturer Information |
|  Batch Code |  Expiry Date |  Refer to User Guide |  Complies with European product regulations. 1984: Notified body number |
|  Recycling Information |  Usage Time After Opening the Cap |  GHS03 Oxidizing | |

MANUFACTURER



GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: +90 216 364 15 00 Email: export@gbl.com.tr Web: www.gbl.com.tr