

INSTRUCTION FOR USE Medical Device Surface Cleaner Disinfectant; Alcohol-free



PRODUCT NAME : Medical Device Surface Cleaner Disinfectant;
Alcohol-free

TRADEMARK : GBL® SURFACTO™ QAS

PRODUCT REF NO : 6007

PACKING TYPE	ORDER NO
HDPE; 1 Liter Bottle with spray	436007051000
HDPE; 5 Liter Canister	436007015000

INTENDED USE

Alcohol-free rapid cleaner disinfectant based on quaternary ammonium compounds. Especially suitable for alcohol-sensitive equipment and medical devices.

PRODUCT SPECIFICATION

Form	Liquid
Water Solubility	Soluble
Color	Colorless
Odor	Odorless
pH @ 25°C (ca)	7.2±0.2

COMPOSITION

Didecyl dimethyl ammonium chloride (0,06 %), alkyl dimethyl benzyl ammonium chloride (0,1 %), chlorhexidine digluconate (0,11 %), stabilizing agents, corrosion inhibitor, other auxiliary agents, distilled water. Does not contain any phenol derivatives, aldehyde, or phosphate.

MICROBIOLOGICAL PROPERTIES

Effect	European Standard	Microorganisms (with ATCC)	Concentration of use	Contact time
Bactericidal	EN 13727	<i>Staphylococcus aureus</i> ATCC 6538 <i>Pseudomonas aeruginosa</i> ATCC 15442 <i>Enterococcus hirae</i> ATCC 10541	Ready-to-use	1 min.
Fungicidal	EN 13624	<i>Candida albicans</i> ATCC 10231 <i>Aspergillus brasiliensis</i> ATCC 16404	Ready-to-use	5 min.
Mycobactericidal	EN 14563	<i>Mycobacterium terrae</i> ATCC 15755	Ready-to-use	1 min.

HOW TO USE

Ready-to-use. Spray on the medical device surface. Wait for 1 minute for bactericidal effectiveness. Dry with a sterile wipe. Does not leave a trace or a greasy film.

STABILITY AND STORAGE










Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/25 °C	2 years	3 months

PACKAGING

Product	Packaging Type	Packaging Units
GBL® SURFACTO™ QAS	HDPE; 1 Liter Bottle with spray	1L x 20
GBL® SURFACTO™ QAS	HDPE; 5 Liter Canister	5L x 2

LABEL SYMBOL KEYS

 REF	Catalog Number	 Production Date	 Storage Temperature	 Manufacturer Information
 LOT	Batch Code	 Expiry Date	 Refer to User Guide	 Recycling Information
	Usage Time After Opening the Cap			

MANUFACTURER



GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: +90 216 364 15 00 **Email:** export@gbl.com.tr **Web:** www.gbl.com.tr