

## INSTRUCTION FOR USE

### Hemodialysis Machine Surface Cleaner Disinfectant



**PRODUCT NAME** : Hemodialysis Machine Surface Cleaner Disinfectant  
**TRADEMARK** : GBL® SURFACTO™ NEUTRAL  
**PRODUCT REF NO** : 6009

PACKING TYPE	ORDER NO
HDPE; 1 Liter Bottle with spray	436009051000
HDPE; 5 Liter Canister	436009015000

### **INTENDED USE**

Hemodialysis machines surface cleaning and disinfection between treatments. Suitable for efficient cleansing of dialysis machine surfaces.

### **PRODUCT SPECIFICATION**

Form	Liquid
Water Solubility	Soluble
Color	Colorless
Odor	Odorless
pH @ 25°C (ca)	7.2±0.2

### **COMPOSITION**

Alkyl dimethyl benzyl ammonium chloride (0,1%), dodecyl trimethyl ammonium bromide, propylene glycol, benzotriazole, ethanol 96%, stabilizing agent, and distilled water. Free of glutaraldehyde, formaldehyde, buffers, and other phenol derivatives.

### **MICROBIOLOGICAL PROPERTIES**

Effect	European Standard	Microorganisms (with ATCC)	Concentration of use	Contact time
Bactericidal	EN 13727	Staphylococcus aureus 6538 Pseudomonas aeruginosa 15442 Enterococcus hirae 10541	Ready-to-use	1 min.
Fungicidal	EN 13624	Candida albicans 10231	Ready-to-use	5 min.
		Aspergillus brasiliensis ATCC 16404	Ready-to-use	60 min.

### **HOW TO USE**

Ready-to-use. Spray on the medical device surface. Wait for 1 minute for bactericidal efficiency.

Wipe with a sterile wipe.

## **WARNINGS**

Flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces. No smoking. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin.

## **STABILITY AND STORAGE**










Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
15/25 °C	2 years	3 months

## **PACKAGING**

Product	Packaging Type	Packaging Units
GBL® SURFACTO™ NEUTRAL	HDPE; 1 Liter Bottle with spray	1L x 20
GBL® SURFACTO™ NEUTRAL	HDPE; 5 Liter Canister	5L x 2

## **LABEL SYMBOL KEYS**

 REF	Catalog Number		Production Date		Storage Temperature		Manufacturer Information
 LOT	Batch Code		Expiry Date		Refer to User Guide		Usage Time After Opening the Cap
	Recycling Information						

## **MANUFACTURER**



**GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi**

**HQ:** Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

**Factory:** Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

**Phone:** +90 216 364 15 00 **Email:** [export@gbl.com.tr](mailto:export@gbl.com.tr) **Web:** [www.gbl.com.tr](http://www.gbl.com.tr)