

INSTRUCTION FOR USE

Glutaraldehyde Activator



PRODUCT NAME : Glutaraldehyde Activator
TRADEMARK : Rosa-Chem GA ACT
PRODUCT REF NO: 6016

PACKING TYPE	ORDER NO
125 ml HDPE; Bottle	436016000125

INTENDED USE: It is used to increase alkaline pH and to increase sporicidal activity in high-level disinfection and sterilization processes with glutaraldehyde.

PRODUCT SPECIFICATION

Form	Liquid
Color	Colorless
Odor	Odorless
pH @ 25°C	10.0±0.5

COMPOSITION

Buffer, preservative and distilled water.

HOW TO USE

Ready to use. Add 25 ml of activator for 1 liter glutaraldehyde disinfectant activation (125 ml activator is for 5 liters of disinfectant). Mix slowly. The mixed solution is ready for high-level disinfection and sterilization.

STABILITY AND STORAGE









Rosa-Chem GA ACT should be stored in its original container at a controlled room temperature 5-25°C in a well-ventilated, low-traffic area. Keep the container always sealed and store it in an upright position.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/25 °C	2 years	3 months

PACKAGING

Product	Packaging Type	Packaging Units
Rosa-Chem GA ACT	HDPE; Bottle	125 ml HDPE; Bottle X 108 pcs

LABEL SYMBOL KEYS

 REF	Catalog Number		Production Date		Storage Temperature		Manufacturer Information
 LOT	Batch Code		Expiry Date		Refer to User Guide		Recycling Information

MANUFACTURER**GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi****HQ:** Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE**Factory:** Dudullu OSB Mah. IMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE**Phone:** +90 216 364 15 00 **Email:** export@gbl.com.tr **Web:** www.gbl.com.tr