

INSTRUCTION FOR USE

Medical Equipment Lubricator and Protector



PRODUCT NAME : Medical Equipment Lubricator and Protector
TRADEMARK : LUBRISERVE™
PRODUCT REF NO : 6130

PACKING TYPE	ORDER NO
HDPE; 100 ml Bottle with spray	43613000100
HDPE; 1 Liter Bottle with spray	436130051000

INTENDED USE: It is used to protect medico-surgical instrument joints from corrosion by lubrication.

PRODUCT SPECIFICATION

Form	Liquid
Color	Colorless
pH @ 25 °C (ca)	6,0 +/- 0,4

COMPOSITION

Liquid parafin, antioxidant, butyl glycol.

INDICATIONS

It is suitable for all medical devices which do not have any risk of lubrication.

CONTRAINDICATIONS

It is used for the protection and lubrication of joints in medical devices before use. Use other than this may cause damage to medical equipment.

PRODUCT CONCENTRATION

Ready-to-use. Do not dilute.

HOW TO USE

Spray onto the precious medico-surgical instrument joints after the cleaning and disinfection process.

WARNINGS AND POTENTIAL ADVERSE EFFECTS

Warning. Causes serious eye irritation. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

STABILITY AND STORAGE










Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/25 °C	2 years	3 months

PACKAGING

Product	ORDER NO	Packaging Type	Packaging Units
LUBRISERVE™	436130000100	HDPE; Bottle with spray	100 mL x 100 Bottle
LUBRISERVE™	436130051000	HDPE; Bottle with spray	1L x 20 Bottle

LABEL SYMBOL KEYS

 REF	Catalog Number		Production Date		Storage Temperature		Manufacturer Information
 LOT	Batch Code		Expiry Date		Refer to User Guide		Recycling Information
	GHS07 Harmful						

MANUFACTURER



GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: 0216 364 15 00 Email: export@gbl.com.tr Web: www.gbl.com.tr