

INSTRUCTION FOR USE

Instrument Super Cleaner; Concentrated



PRODUCT NAME : Instrument Super Cleaner; Concentrated

TRADEMARK : WARECLEAN EC PENTA

PRODUCT REF NO : 6159

PACKING TYPE	ORDER NO
0,5 Liter HDPE; Dosing Bottle	436159140500
1 Liter HDPE; Dosing Bottle	436159131000
5 Liter HDPE; Canister	436159025000

INTENDED USE

WARECLEAN EC PENTA provides superior cleaning and pre-disinfection of medical-surgical instruments, medical devices, thermosensitive instruments, and endoscopes. It contains in powerful combination of enzymes (amylase, cellulase, lipase, mannanase and protease) that accelerate soil breakdown. It can be used in manual, ultrasonic, and automatic washing machines and washing tunnels with its low foam formula.

PRODUCT SPECIFICATION

Form	Liquid
Color	Red
Odor	Pine
pH @ 25°C	7.2±0.3

COMPOSITION

Enzymatic complex (amylase, lipase, protease, cellulase, mannanase), glycerol, isopropanol, ethanol (96%), EDTA tetrasodium salt, urea, trisodium phosphate, sodium hydroxide, propylene glycol, butyl glycol, corrosion inhibitor, surfactants, triethanolamine, and RO water.

CONTRAINDICATIONS

It should not be used in devices that use phosphate-free and neutral detergents.

PRECAUTIONS

- Protective gloves (butyl rubber, nitrile rubber, polyethylene, or double-gloved latex), eye protection, face masks, and liquid-proof gowns should be worn when cleaning and sterilizing/disinfecting soiled devices and equipment.
- The outer surface of contaminated, reusable medical devices should be wiped with a cloth.
- The user **MUST** adhere to the Instruction for Use since any modification will affect the effectiveness of the cleaner.

HOW TO USE

1- Pre-cleaning:

Wipe down the external surface of the surgical instrument with gauze or a cloth, to remove the mucous membrane, blood, and filth that adhere to the external surface. Either moistened gauze or dry gauze is fine.

2- Cleaning: Dilute with hot or cold tap water. Depending on the protein residue degree, the dilution rate is between 0,25% and 1%. For very dirty instruments 4 pumps (20mlx4) for 8 Liters of water. For relatively clean instruments 1 pump (20mlx1) for 8 Liters of water. Immerse the instruments into the container. Wait for 5-15 minutes. Scrub the instruments gently with a soft brush. Rinse thoroughly with distilled water (sterile distilled water is preferred). Wipe thoroughly and dry with an absorbent cleaning path. Sterilize the instruments or immerse them to a new soaking bath full of high-level disinfectant. it is used for ultrasonic bath; at 45°C, 1 minute at 0.5%. New ready-to-use solutions must be prepared every day.

WARNINGS AND POTENTIAL ADVERSE EFFECTS

Causes severe skin burns and eye damage. Do not breathe dust/fumes/gas/mist/vapors/spray. Wear protective gloves/ protective clothing/eye protection/face protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF INHALED: Remove the person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to continue rinsing.

Long-term skin contact may cause minor irritation with itching. There is no evidence of adverse reproductive or carcinogenic health.

STABILITY AND STORAGE

Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/35 °C	2 years	3 months

PACKAGING

Product	ORDER NO	Packaging Type	Packaging Units
WARECLEAN EC PENTA	436159140500	0,5 Liter HDPE; Dosing Bottle	1L x 24
WARECLEAN EC PENTA	436159131000	1 Liter HDPE; Dosing Bottle	1L x 12
WARECLEAN EC PENTA	436159025000	5 Liter HDPE; Canister	5L x 2

LABEL SYMBOL KEYS



Catalog Number



Production Date



Storage Temperature



Manufacturer Information



Batch Code



Expiry Date



Refer to User Guide



Recycling Information



GHS05
Corrosive

MANUFACTURER



GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. IMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: 0216 364 15 00 **Email:** export@gb.com.tr **Web:** www.gbl.com.tr