

INSTRUCTION FOR USE

Blood, Corrosion and Rust Remover; Concentrated



PRODUCT NAME : Blood, Corrosion and Rust Remover; Concentrated

TRADEMARK : WARECLEAN™ RC

PRODUCT REF NO : 6170

PACKING TYPE	ORDER NO
1 Liter HDPE; Dosing Bottle	436170131000
5 Liter HDPE; Canister	436170025000

INTENDED USE: Removes blood residues, corrosion, rust, and stains on the medico-surgical instruments. Routine use of WARECLEAN™ RC will increase the life and enhance the efficiency of stainless-steel instruments.

PRODUCT SPECIFICATION

Form	Liquid
Color	Colorless

COMPOSITION

Non-ionic surfactant, phosphoric acid, butyl glycol, isopropyl alcohol, stabilizing agent, and distilled water.

PRODUCT CONCENTRATION

Used dilution: 1 to 49 (20 ml concentrated product into 980 ml water)

HOW TO USE

Dilute with distilled water with a 1/50 ratio (1 x 20 ml pump concentrated product into 980 ml water). Immerse medico-surgical instruments into the prepared solution. Wait for 15-30 minutes depending on the blood residues, corrosion, rust, and stains degree. Rinse thoroughly with distilled water.

WARNINGS AND POTENTIAL ADVERSE EFFECTS

Danger. Causes severe skin burns and eye damage. Do not breathe dust/fumes/gas/mist/vapors/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/ shower. IF INHALED: Remove the person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to continue rinsing. Store locked up.

STABILITY AND STORAGE

Product; In the shade, in the original packaging, the package should be kept tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
---------------------	------------	----------------------------------

15/25 °C











2 years

3 months

PACKAGING

Product	ORDER NO	Packaging Type	Packaging Units
WARECLEAN™ RC	436170131000	1 Liter HDPE; Dosing Bottle	1L x 12
WARECLEAN™ RC	436170025000	5 Liter HDPE; Canister	5L x 2

LABEL SYMBOL KEYS

 REF	Catalog Number		Production Date		Storage Temperature		Manufacturer Information
 LOT	Batch Code		Expiry Date		Refer to User Guide		Recycling Information
	Usage time after opening the cap		GHS05 Corrosive				

MANUFACTURER**GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi**

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. IMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: 0216 364 15 00 Email: export@gb.com.tr Web: www.gb.com.tr