

INSTRUCTION FOR USE

OPA Neutralizer



PRODUCT NAME : OPA Neutralizer
TRADEMARK : ROSA- CHEM OPA
PRODUCT REF NO : 6218

PACKING TYPE	ORDER NO
White HDPE; Bottle	436218000032

INTENDED USE: OPA Neutralizer is used for neutralization of the disinfectant containing ortho-phthalaldehyde. Ortho-phthalaldehyde can be dangerous for aquatic life (H400: Very toxic to aquatic life. H410: Very toxic to aquatic life with long-lasting effects). ROSA-CHEM OPA is a chemical agent to neutralizes ortho-phthalaldehyde side effects and makes it safe for the environment, especially aquatic life. After neutralization, it can be discharged to the sewage system as household waste. If not neutralized, it must be discharged as chemical waste.

PRODUCT SPECIFICATION

Form	Crystalline
Color	White
Odor	Odorless
CAS No	56-40-6
EC No	200-272-2

COMPOSITION

Glycine.

HOW TO USE

For 5-liter solution use 1 bottle / 32 grams neutralizer. Use protective equipment. Open the neutralizer bottle carefully. Pour the whole neutralizer bottle into the disinfection container. Stir 10 minutes. After 10 minutes you can discharge safely the neutralized solution according to the waste disposal rules of your country. Neutralized solution is not harmful to the environment.

WARNINGS AND POTENTIAL ADVERSE EFFECTS

IF SWALLOWED: Rinse mouth. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to continue rinsing.

STABILITY AND STORAGE









Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/25 °C	2 years	3 months

PACKAGING

Product	ORDER NO	Packaging Type	Packaging Units
ROSA- CHEM OPA	436218000032	White HDPE; Bottle	32 g X 120

LABEL SYMBOL KEYS

 REF	Catalog Number		Production Date		Storage Temperature		Manufacturer Information
 LOT	Batch Code		Expiry Date		Refer to User Guide		Recycling Information

MANUFACTURER



GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: 0216 364 15 00 Email: export@gbl.com.tr Web: www.gbl.com.tr