

## INSTRUCTION FOR USE

### OPA Neutralizer



**PRODUCT NAME** : GA Neutralizer  
**TRADEMARK** : ROSA- CHEM GA  
**PRODUCT REF NO** : 6219

PACKING TYPE	ORDER NO
White HDPE; Bottle	436219000125

**INTENDED USE:** ROSA-CHEM GA is used to neutralize harmful-to-the environment effects of aldehyde active ingredient high-level disinfectant solution. After neutralization, disinfectant solution can be discharged to the sewage system as household waste. If not neutralized, it must be discharged as chemical waste.

### PRODUCT SPECIFICATION

Form	Powder
Color	White
Odor	Pungent
pH @ 25 °C (ca)	4,5 at 50 g/l at 20 °C
CAS No	7681-57-4
EC No	231-673-0

### COMPOSITION

Sodium metabisulfite.

### HOW TO USE

For 5-liter solution use 1 bottle / 125 grams neutralizer. Use protective equipment. Open the neutralizer bottle carefully. Pour the whole neutralizer bottle into the disinfection container. Stir 10 minutes. After 10 minutes you can discharge safely the neutralized solution according to the waste disposal rules of your country. Neutralized solution is not harmful to the environment.

### WARNINGS AND POTENTIAL ADVERSE EFFECTS

**Danger.** Harmful if swallowed. Causes serious eye damage. Wear protective gloves/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. EUH031: Contact with acids liberates toxic gas.

### STABILITY AND STORAGE

Store in its original packaging, without direct sunlight, in a shady environment. Keep it tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
5/25 °C	2 years	3 months

## PACKAGING

Product	ORDER NO	Packaging Type	Packaging Units
ROSA- CHEM GA	436219000125	White HDPE; Bottle	125 gr x 108

## LABEL SYMBOL KEYS

Catalog  
NumberProduction  
DateStorage  
TemperatureManufacturer  
Information

Batch Code



Expiry Date

Refer to  
User GuideRecycling  
Information**GHS05**  
Corrosive**GHS07**  
Harmful

## MANUFACTURER

**GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi**

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: 0216 364 15 00 Email: [export@gbl.com.tr](mailto:export@gbl.com.tr) Web: [www.gbl.com.tr](http://www.gbl.com.tr)