

INSTRUCTION FOR USE

Concentrated Disinfectant for Treatment of Aspiration Systems

PRODUCT NAME : Concentrated Disinfectant for Treatment of Aspiration Systems

TRADEMARK : LIQUISEPT™ ASPIRATION SA

PRODUCT REF NO: 6818



PACKING TYPE	ORDER NO
500 ml HDPE; Dosing Bottle	436818140500
1 Liter HDPE; Dosing Bottle	436818131000
5 Liter HDPE; Canister	436818025000

INTENDED USE: Used for cleaning detergent solution for the daily treatment of all types of suction systems, amalgam separators and spittoons.

PRODUCT SPECIFICATION

Form	Liquid
Color	Yellowish
Odor	Specific
pH @ 25°C (ca):	7.5±0.3

COMPOSITION

Didecyl dimethyl ammonium chloride (6%), alkyl dimethyl benzyl ammonium chloride (5%), isopropanol (5%), detergents, chelating agents, defoamer, thickening agents, emulsion agents, protease, buffers, corrosion inhibitors and stabilizing agent. Without aldehyde and phenol.

MICROBIOLOGICAL PROPERTIES

Effect	Standard	Microorganism (with ATCC)	Concentration of use	Contact Time
Bactericidal	EN 13727 / EN 14561	<i>Staphylococcus aureus</i> ATCC 6538 <i>Pseudomonas aeruginosa</i> ATCC 15442 <i>Enterococcus hirae</i> ATCC 10541	1/100	15 min.
Fungicidal	EN 13624 / EN 14562	<i>Candida albicans</i> ATCC 10231 <i>Aspergillus brasiliensis</i> ATCC 16404	1/100	15 min. 60 min.

*The product is the activity table with the specified concentration and duration of use.

HOW TO USE

At the end of the day, prepare 2 liters of diluted solution using Liquesept Aspiration SA. Dilute at 1% (20 ml for 2 liter water). Fill the dosing to 20 ml indicator on the bottleneck. For 5 liter can, press 1 pump. Gently shake the Liquesept Aspiration SA to mix the solution. Attach to the suction system and aspirate the diluted solution. To achieve efficacy, a contact time of at least 15 minutes is required. The solution can be left in the system overnight. To complete the treatment, fill the Liquesept Aspiration SA with at least 1 liter of clean water and aspirate to flush the system.

WARNINGS AND POTENTIAL ADVERSE EFFECTS

Causes severe skin burns and eye damage. Do not breathe dust/fumes/gas/mist/vapors/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF INHALED: Remove the person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do continue rinsing.

STABILITY AND STORAGE

Product; in the shade, in the original packaging, the package should be kept tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
15/25 °C	2 years	3 months

PACKAGING

Product	Packaging Type	Packaging Units
LIQUISEPT™ ASPIRATION SA	500 ml HDPE; Dosing Bottle	500 ml x 24
LIQUISEPT™ ASPIRATION SA	1 Liter HDPE; Dosing Bottle	1L x 12
LIQUISEPT™ ASPIRATION SA	5 Liter HDPE; Canister	5L x 2

LABEL SYMBOL KEYS



Catalog
Number



Production
Date



Storage
Temperature



Manufacturer
Information



Batch Code



Expiry Date



Refer to User
Guide



Recycling
Information



GHS05
Corrosive

MANUFACTURER

GBL® Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Anonim Şirketi

HQ: Serifali Mah. Hattat Sk. No:10 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Factory: Dudullu OSB Mah. İMES C Blok 305 Sk. No:16 P.O.: 34775 Ümraniye İstanbul TÜRKİYE

Phone: +90 216 364 15 00 **Email:** export@gbl.com.tr **Web:** www.gbl.com.tr



